

RECOMMENDATION FOR SANITIZATION OF OTICON PRODUCTS IN CLASSROOMS



OTICON EDUMIC

These recommendations apply to the following Oticon products when shared between users:

- Oticon Edumic
- Oticon Amigos

A surface disinfectant such as a solution with 70% isopropanol should be used to disinfect the Oticon Mic.

When a student passes their Oticon Mic to their teacher, the mic should be sanitized.

- The teacher should wash their hands with hand sanitizer prior to starting the sanitization process.
- Spray a piece of paper towel with the isopropanol disinfectant.
- Wipe the device off with the disposable paper towel.
- Never directly spray disinfectant directly onto the device.
- Make sure openings, in particular microphone openings, are never in contact with liquid as this may damage electronic components.
- While wiping the device with disinfectant, do not press any buttons as long as the device is still wet.
- Classroom teachers will be provided with their own lanyard for the Oticon Mic by the APSEA Itinerant teacher, as needed.
- Attach the lanyard to the Oticon transmitter mic and place around neck.
- Turn Oticon Mic on.
- Wash hands with hand sanitizer after the sanitization process is complete.
- At the end of the class, remove the lanyard from the Oticon Mic and repeat the cleaning process. Return the Oticon Mic to the student.



OTICON AMIGO