

# RECOMMENDATION FOR SANITIZATION OF ROGER PRODUCTS IN CLASSROOMS



ROGER  
TOUCHSCREEN MIC



ROGER INSPIRO MIC

**These recommendations apply to the following Roger products when shared between users:**

- Roger Touchscreen Mic
- Roger Inspiro Mic
- Roger Pass-around Mics

**A surface disinfectant such as a solution with 70% isopropanol should be used to disinfect the Roger Mic.**

**When a student passes their Roger Mic to their teacher, the mic should be sanitized.**

- The teacher should wash their hands with hand sanitizer prior to starting the sanitization process.
- Spray a piece of paper towel with the isopropanol disinfectant.
- Wipe the device off with the disposable paper towel.
- Never directly spray disinfectant directly onto the device.
- Make sure openings, in particular microphone openings, are never in contact with liquid as this may damage electronic components.
- While wiping the device with disinfectant, do not press any buttons as long as the device is still wet.
- Classroom teachers will be provided with their own lanyard for the Roger Mic by the APSEA Itinerant teacher.
- Attach the lanyard to the Roger transmitter mic and place around neck.
- Turn Roger Mic on.
- Wash hands with hand sanitizer after the sanitization process is complete.
- At the end of the class, remove the lanyard from the Roger Mic and repeat the cleaning process. Return the Roger Mic to the student.